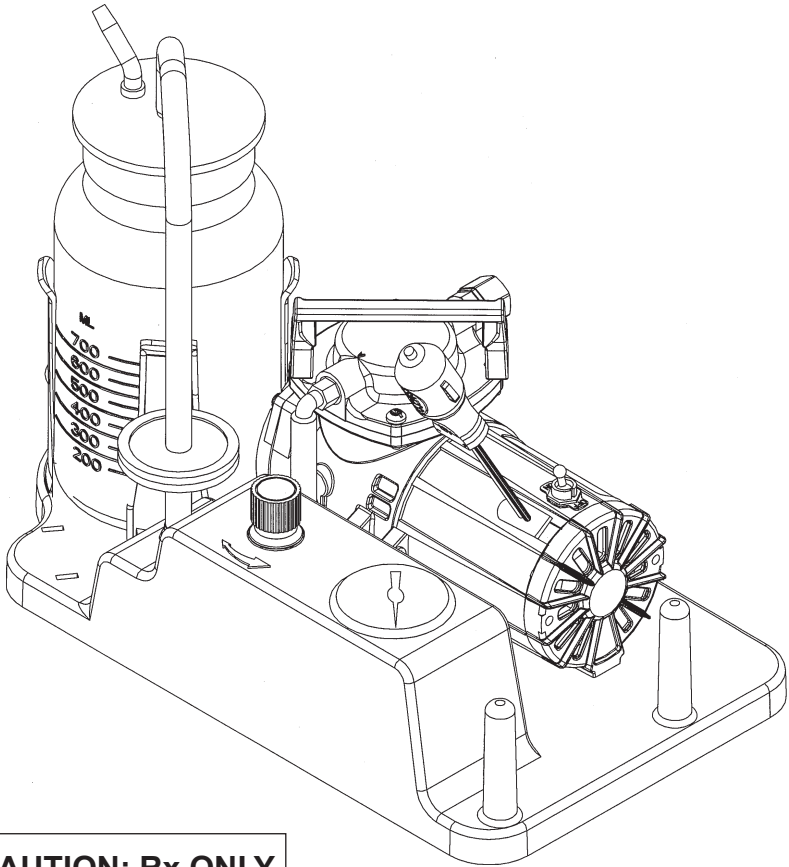
THOMAS

A Gardner Denver Product

MEDI-PUMP®

**Aspirator Model
1634DC/GL**

CAUTION: Rx ONLY

READ ALL INSTRUCTIONS BEFORE USING THIS DEVICE

Important Safeguards

This Medi-Pump product is an electrical device. Always follow basic safety precautions when using this device, especially when children are present. The following important information is highlighted by these terms:

DANGER- Urgent safety information for hazards that will cause serious injury or death.

WARNING- Important safety information for hazards that might cause serious injury.

CAUTION- Information for preventing damage to the product.

NOTE- Information to which you should pay special attention.

SYMBOLS

ATTENTION - Followed by important information

WARNING: THIS PRODUCT SHOULD NOT BE OPERATED UNLESS THE OPERATOR HAS BEEN INSTRUCTED BY A PHYSICIAN OR OTHER QUALIFIED HEALTH CARE PROFESSIONAL.

Important Safeguards

SAVE THESE INSTRUCTIONS

DANGER- To reduce the risk of electrocution:

- 1 Always unplug this product immediately after using.
- 2 Do not use while bathing.
- 3 Do not place or store product where it can fall or be pulled into a tub or sink.
- 4 Do not place in or drop into water or other liquid.
- 5 Do not reach for a product that has fallen into water. Unplug immediately.

WARNING- To reduce the risk of burns, electrocution, fire, or injury to persons.

- 1 Never leave this product unattended when plugged in.
- 2 Use this product only for its intended use as described in the operator manual. Do not use attachments not recommended by the manufacturer.
- 3 Closely monitor and supervise use by, on, or near invalids and children.
- 4 Never operate this product if:
 - a. It has a damaged power cord or plug.
 - b. It is not working properly.
 - c. It has been dropped or damaged.
 - d. It has been dropped into water.

Return the product to an authorized Thomas Industries service center or distributor.

- 5 Keep the power cord away from heated surfaces.
- 6 Never use while sleeping or drowsy.
- 7 Never drop or insert any object into any opening or hose.
- 8 Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.
- 9 Never block the air openings of the device or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.

DANGER- The Medi-Pump aspirator is a vacuum suction device designed for the collection of nonflammable fluid materials in medical applications only. Improper use during medical applications can cause injury or death.

Table Of Contents

Introduction	5
Specifications	5
Important Parts of the Device	6
Set-Up	7
Operating Instructions	7
Cleaning the Device	8
Replacement Parts	9-10
Filter and Muffler Replacement	11
Troubleshooting Guide	12
Maintenance	13
Worldwide Service Centers	14-15
Warranty Information	16

Introduction

Purpose of Device

The Medi-Pump aspirator is a vacuum suction device to remove body fluids from a patient's airway or respiratory system.

Description of Device

The Medi-Pump aspirator was created for home care use. It was designed with a high performance vacuum pump. That results in strong suction with shorter vacuum level recovery times. The device has a built-in vacuum regulator and gauge in a non-rusting base. The jar holder will accommodate a 800cc size glass collection jar. (P/N 624380)

Specifications

SIZE

14.75" L x 8.25" W x 10.01" H
(375mm x 255mm x 210mm)

WEIGHT

15 (6.8 kilograms)

ELECTRICAL REQUIREMENTS

Model No.1634DC/GL 12VDC, 6.0A

OPERATING TEMPERATURE

Ambient temperature range
104°F max./50°F min.
(40°C max./10°C min.)

Storage:

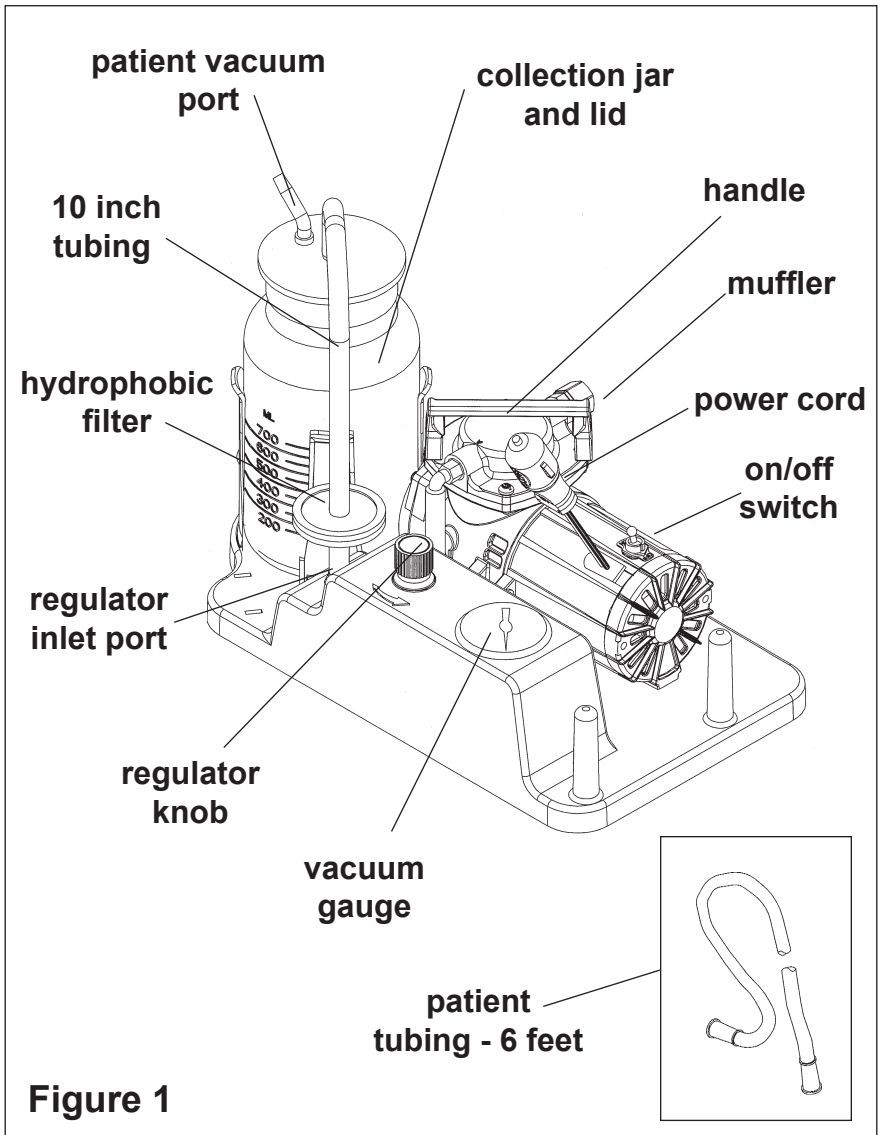
Room temperature preferred

Color:

Grey and white case with clear collection jar.

Important Parts of the Medi-Pump Device

Familiarize yourself with the important parts shown below before set-up.

Set Up

Refer to FIGURE 1

- 1 Set the device on a level, sturdy surface so the controls can be easily reached and adjusted.
- 2 Connect one end of the tubing to the lid vacuum port.

Note

The in-line hydrophobic filter is designed to prevent foreign material from being drawn into the device.

Caution

Do not move this device with a full collection jar.

Operation Instructions

Refer to FIGURE 1

- 1 Be sure that the power switch is in the off (down) position. Plug in the device and turn the switch to the ON (up) position.
- 2 Block the patient tube leading from the collection jar and allow the gauge to reach a stable vacuum level.
- 3 Set the vacuum level by turning the regulator knob counter-clockwise to increase vacuum or clockwise to decrease vacuum.
- 4 Your physician will have prescribed the use of a special vacuum tool to aid suction. Push this tool onto the 6 foot long patient tubing as instructed by your qualified health care professional.

Warning

Use the device the way it was intended and as instructed by your physician or qualified health care professional only.

Warning

Use the vacuum setting directed by your physician or qualified health care professional only. Using a higher setting than instructed may lead to a life threatening situation.

NOTE: This procedure should be performed each time you use this device.

Cleaning the Device

- 1 Make sure the ON/OFF switch is in the OFF (down) position.
- 2 Unplug the device from the wall outlet and allow suction level to drop.
- 3 Remove the patient tubing from the lid vacuum port. Tubing must be cleaned after every use. Run hot tap water through it. Soak in a solution of one part vinegar to three parts hot water for 20 minutes. Rinse with hot tap water and air dry.
- 4 Remove the tubing from the top of the jar lid.
- 5 Remove the jar and lid assembly from the holder. Place on a firm, flat surface. Lift the lid release tabs and remove the gasket and lid from the jar.
- 6 Wash the jar, gasket, lid and tube in a hot water/dishwashing detergent solution and rinse with hot tap water. Soak in a solution of one part vinegar to three parts hot water for 20 minutes. Rinse with hot tap water and air dry.
- 7 Wipe the outside of the case with a clean, damp cloth after each use.

NOTE
The collection jar must be emptied after each use.

CAUTION
Jar, lid, gasket and tubing are dishwasher safe.

CAUTION
Never submerge this device in water, hold under running water or spray with water or other liquids. This will result in damage to the device.

Replacement Parts

The following items may deteriorate over time. You will need to inspect and replace them to keep the device operational, and to maintain proper sanitary conditions.


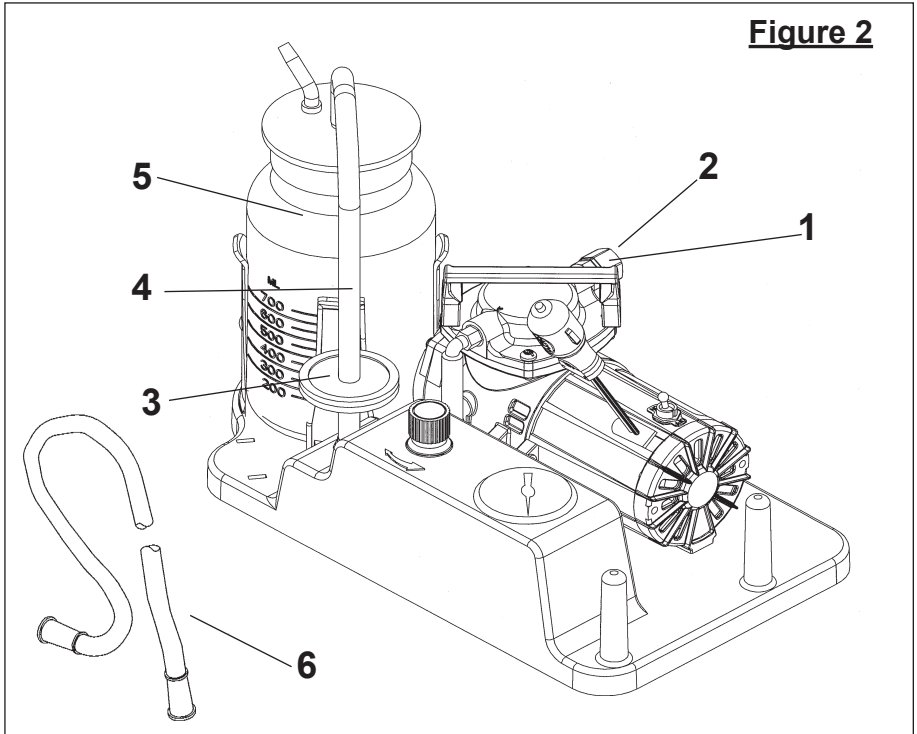
Figure 2

Figure 2 - Parts List

- | | | |
|----------|-------------------|--------------------------------|
| 1 | P/N 626492 | Muffer Cover |
| 2 | P/N 641292 | Muffer Filter |
| 3 | P/N 641130 | Hydrophobic Filter |
| 4 | P/N 618254 | 10 Inch Tubing |
| 5 | P/N 620573 | Jar & Lid Assembly |
| 6 | P/N 615725 | Patient Tubing - 6 Feet |

Vacuum Pump Replacement Parts

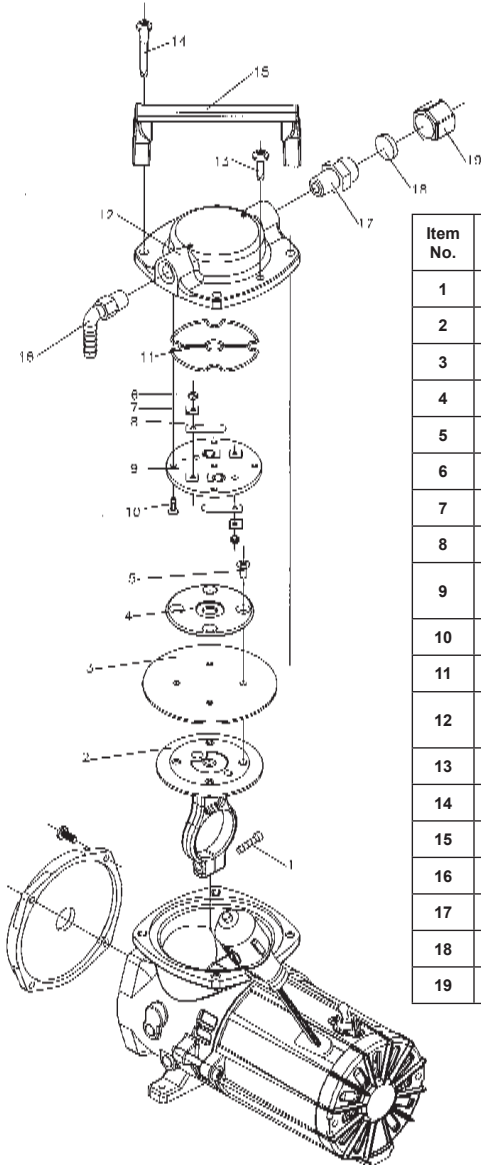The following items are service parts and should be replaced by a qualified service center or a distributor.

Item No.	Part No.	Description	Qty.
1	625114	Connecting Rod Screw	1
2	607139	Connecting Rod	1
3	608144	Diaphragm	1
4	654649	Hold Down Plate	1
5	625540	Hold Down Plate Screw	4
6	625160	Flapper Valve Screw	2
7	617045	Valve Keeper Strip	2
8	621102	Valve Flapper	2
9	625606	Valve Plate to Head Screw	4
10	654129	Valve Plate	1
11	633439	Head Gasket	1
12	661219-504	Head	1
13	625492	Head Screw	5
14	624704	Elbow Fitting - Barbed	1
15	629113	Handle	1
16	625493	Handle Screw	2
17	624676	Muffler Fitting	1
18	641292	Muffler Filter	1
19	626492	Muffler Cover	1

Muffler and Filter Replacement

Muffler Replacement

The muffler should be replaced every three months or if contaminated. See page 9, Figure 2, items 1 and 2.

- 1 Grasp the muffler cover firmly and unscrew it.
- 2 Lift out the old muffler with a tweezers and replace with the new muffler.
- 3 Screw the muffler cover back into the muffler fitting.

Hydrophobic Filter Replacement

Change the filter whenever foreign material is drawn into it or if you notice a reduction in vacuum level. See Page 9, Figure 2, items 3 and 4.

- 1 Disconnect the 10" tubing from the barbed fitting on the top of the filter.
- 2 Gently pull the hydrophobic filter off of the 2 inch piece of plastic tubing from the regulator inlet port.
- 3 With the inlet facing down, push the new filter onto the 2 inch plastic tubing from the regulator port.
- 4 Push the 10" tubing onto the barbed fitting on top of the filter.

Note

If the muffler cover is dusty or clogged, hold it under running tap water to clean. Air dry.

Caution

The hydrophobic filter is designed to prevent foreign material from being drawn into the device. Operating it without the hydrophobic filter may allow fluids to enter the device causing damage to its' internal components.

Troubleshooting

Symptoms	Possible Causes	Remedies
Reduced/low vacuum.	<ol style="list-style-type: none"> 1 Clogged hydrophobic filter. 2 Regulator improperly set. 3 Air leak. 4 Jar lid or gasket not secured to jar. 	<ol style="list-style-type: none"> 1 Replace the filter following directions in this manual. 2 Set regulator vacuum level following operating instructions. 3 Check the tubing and connectors for possible leaks. Attach, tighten, or replace. 4 Be sure the jar lid and gasket fit tightly on the jar.
Device does not start.	<ol style="list-style-type: none"> 1 Power cord not plugged in. 2 Defective switch. 3 Vacuum still exists in system. 	<ol style="list-style-type: none"> 1 Plug in the power cord. 2 Contact your service center or distributor. 3 Disconnect tubing from the collection jar to release vacuum. Reconnect tubing and turn on the device.
Gauge reads incorrectly.	Faulty gauge.	Contact service center for repair.
Device runs but gauge reads zero.	Gauge tubing not connected or is leaking.	Check the tubing and connector for leaks. Attach, tighten, or replace.
Device stops running during use.	<ol style="list-style-type: none"> 1 Device overheated. 2 Blown fuse or circuit breaker in household circuit. 3 Low voltage to device. 	<ol style="list-style-type: none"> 1 Unplug device and allow to cool for 10 minutes. Plug in and restart. 2 Replace fuse in DC adapter. 3 Check 12 volt power supply.

Maintenance

All major maintenance on the device must be performed by an authorized Thomas Industries Inc. service center or distributor.

Caution

This device is oil-less, do not lubricate.

Note

It will be necessary for you to provide your model number and serial number when contacting a service center for help.

You will find this information on the nameplate located behind the collection jar. Please take a moment to record this information for future use.

WARNING 
Do not take aspirator case apart. Failure to comply could result in electrical shock. All disassembly must be carried out by an authorized Thomas Industries Inc. service center or distributor.

Aspirator model _____

Serial number _____

Worldwide Service Centers

BOLIVIA

Importadora Fernando
PHONE: 591-3-542525
EMAIL: imp-fernando@cotas.com.bo
Santa Cruz

CHILE

Dipromed S.A.
PHONE: 56-2-4731300
jespinoza@dipromed.cl
Santiago

Indura Ltda.
PHONE: 56-2-530-3054
Santiago

Tecnigen S.A.
PHONE: 56-2-235-8011
Santiago

CHINA (PEOPLE'S REPUBLIC)

New-Sinitic Enterprising Limited
Shanghai
PHONE: (86 21) 5480 0142
E-MAIL: kenchina@sh163.net

COLOMBIA

Imcolmédica S.A.
Santafe De Bogota, D.C.
PHONE: 571-287-1811
E-MAIL: imcolmedica@imcolmedica.com.co

COSTA RICA

Representaciones G.M.G., S.A.
PHONE: 506-232-1164
San José

DOMINICAN REPUBLIC

Farmaco Quimica Nacional
Santo Domingo
PHONE: 809-381-0906
EMAIL: farmaconal@codetel.net.do

ECUADOR

Boelectronica Blanco S.A.
Guayaquil
PHONE: 593-4-881569
EMAIL: blancosa@interactive.net.ec

EL SALVADOR

Oxgasa Oxigeno y Gases de El Salvador
San Salvador
PHONE: 503-234-3200
E-MAIL: oxgasa-gerenciaventas@salnet.net

GUATEMALA

Importadora Jaeger, S.A.
Guatemala City
PHONE: 502-232-2285
EMAIL: contact@jaeger.com.gt

HONG KONG

Clinicon Medical Limited
Tsim Sha Tsui
PHONE: (852) 2730 1883
E-MAIL: clinicon@hkstar.com

INDONESIA

Labora Medical Company
PHONE: 65-271 8038
Maju Jaya

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Hospitalar, S.A.
PHONE: 51-14-45-7922
EMAIL: hospitalar1@amauta.rcp.net.pe
Lima

PHILIPPINES

Progressive Medical Corporation
PHONE: 0 63 2 687-7888
E-MAIL: www.pmc-ph.com
Pasig City 1605

PUERTO RICO

Technical Systems Specialties Corp.
PHONE: 787-753-7961
San Juan

SINGAPORE

Kingston Medical Supplies (Pte.) Ltd.
PHONE: 65 6745 3922
E-MAIL: gefoo@kingsmed.com.sg

Lozon (S) Pte. Limited
PHONE: (65) 299 9123
E-MAIL: lozon@mbox5.singet.com.sg

SOUTH AFRICA

Medical Distributors
PHONE: 011-27-11-314-3000
Johannesburg

SOUTH KOREA

Koh Bong & Company
PHONE: (82 02) 762-4855
E-MAIL: www.kohbong2000@yahoo.co.kr
Seoul

SRI LANKA

Denme Electro Medicals (Pvt) Ltd.
PHONE: (94 1) 699 974
E-MAIL: denme@eureka.lk
Colombo

TAIWAN

Apex Medical Corporation
PHONE: (886 2) 2695 4122
E-Mail: eliza@apexmedical.com.tw
Taipei Hsien

THAILAND

Technical Win Group Co., Ltd.
PHONE: (662) 5592354
E-MAIL: twg@loxinfo.co.th
Bangkok

URUGUAY

Elena Ltda.
PHONE: 59-82-409-0347
EMAIL: asuberb@adinet.com.uy
Montevideo

VENEZUELA

Tecnomed J. Trapp C.A.
PHONE: 58-212-575-2422
Caracas

VIETNAM

H.V. Trading Company Limited
E-MAIL: Phongtv@fpt.vn
PHONE: (84 4) 835 9011
Hanoi

TAN HUNG Co., Ltd.
PHONE: (84 8) 865 4333
FAX: (84 8) 865 4536
Ho Chi Minh City

Warranty Information

Thomas Industries Inc. warrants the device for 18 months from the date of manufacture due to faulty parts or workmanship. Check with your dealer for more details.

This warranty is limited to the original purchaser of the device. Any defective part(s) or assembly will be repaired or replaced, at the sole discretion and determination of Thomas Industries Inc. if the unit has not been misused or tampered with during the warranty period.

Normal maintenance items, as outlined in this manual and disposable components are not covered by this warranty. Shipping charges, if any, shall be paid by the purchaser.

Note

You may determine the date of manufacture by reading the first 4 numbers from the serial number field of the nameplate. These numbers represent the month and year that the device was made. For example, if it reads "0204", the date of manufacture is February of 2004. If it reads "1295" the date of manufacture is December of 1995.

Note

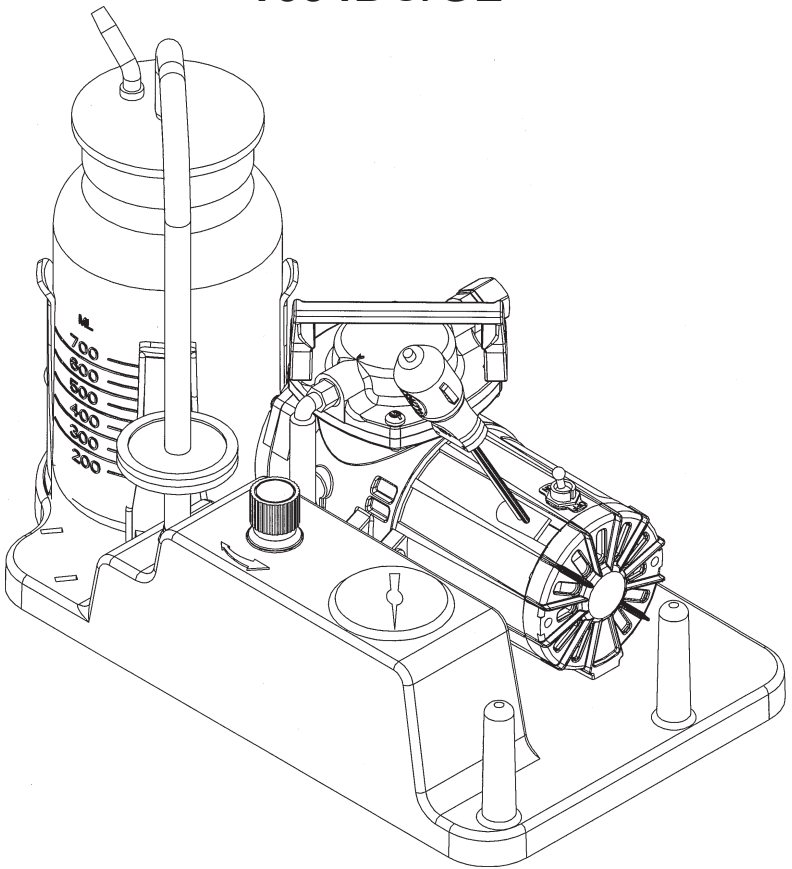
There is no other express warranty. Implied warranties, including those of merchantability and fitness for a particular purpose, are limited to the duration of the express limited warranty and to the extent permitted by law and all implied warranties are excluded. This warranty does not cover providing a loaner compressor, compensation for costs incurred for compressor rental, or for labor costs incurred in repairing or replacing a defective part(s).

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